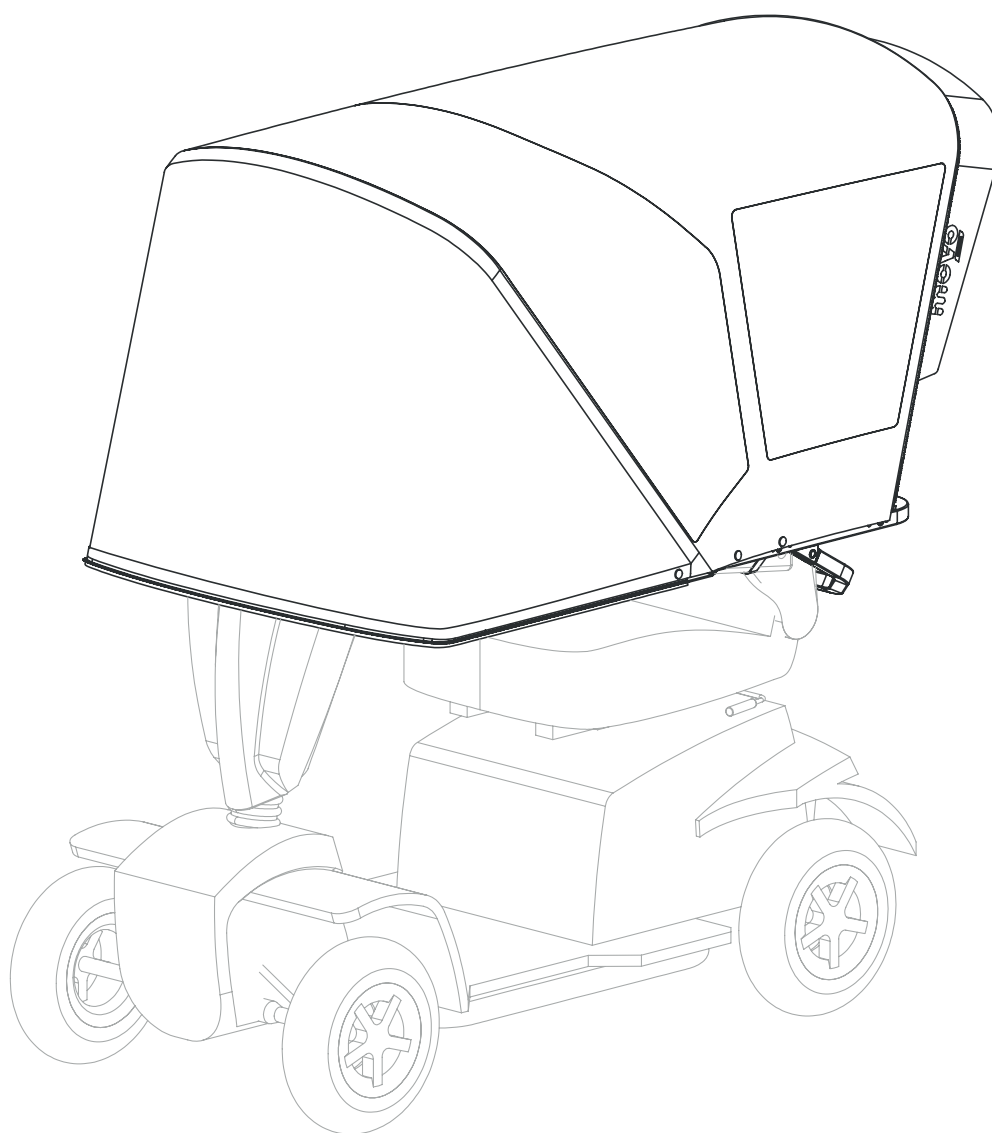
Gebruikershandleiding  
User manual  
Manuel d'utilisation


# moVo

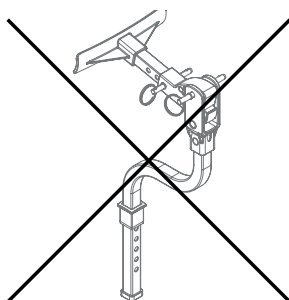evolution

MV-100-2

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Users of scooters with a low backrest (< 51 cm) should not install item 01-021 and 01-022. Page 29 of the manual contains specific advise on ionstalling the roof on this type of seat.

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For more information, visit **[www.skylinemobility.com](http://www.skylinemobility.com)** for a full video that will walk you through the steps.

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gb

## Content

<b>Conditions and limitations</b>	5 - 7
<b>Parts</b>	8 - 9
<b>Setting the driving position</b>	10
<b>Mounting the seat frame</b>	11 - 17
<b>Mounting the canopy frame</b>	18 - 23
<b>Installation canopy</b>	24 - 27
<b>Final check</b>	28 - 30

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## MOVO WARRANTY CONDITIONS AND LIMITATIONS

### Movo Evolution Canopy

**1-year LIMITED WARRANTY:** Please read and understand the instructions before attempting to install the new Movo Evolution canopy.

Warrants to the first retail purchaser that the Movo canopy is free from defects in materials and workmanship for one (1) year from the date of original purchase. This warranty does not apply to damages caused by strong winds or hail, improper installation, misuse, or neglect, use of abrasive or chemical cleaners, overexposure to UV rays, accidents, lack of maintenance, unsuitable environment such as chlorine, alterations or abuse. Although the material used is very resistant, it can tear during installation if not done correctly and Skyline Mobility will not assume any responsibility for such installation. In addition, wear and fading resulting from normal use are excluded from this warranty, as well as discoloration caused by over exposure to UV rays as well as the appearance of white line in the clear plastic.

### UNAUTHORIZED USAGE

- Do not use in high winds, extreme rain, hail, and other unusual weather events.
- Do not use on an electric vehicle with a maximum speed exceeding 25 km/h.
- Do not use on other types of vehicles other than four-wheel or three-wheel electric scooters.
- Do not use at sub-freezing temperatures, below zero degrees celsius.
- Do not repeatedly fold and un-fold at an ambient temperature of less than 7 degrees Celsius, as it becomes more rigid.
- This canopy is not a substitute for a protective storage canvas.
- For some scooter models, the mirrors may need to be "folded-in" to maintain optimum steering control, as the canopy was designed at a width no larger than the average doorframe.
- Never use the product if it creates blind spots while driving. If necessary, readjust the canopy and the position of the seat to reach full visibility.
- Do not use if the canopy obscures the vehicle's lights or reflectors. The user is responsible for its vehicle's visibility.
- Do not fold or unfold the canopy with one hand. Always use two hands and lift both sides at the same time.

The person performing the assembly is responsible for ensuring that the roof installation is correct as defined in the user guide or in the assembly video on the manufacturer-/distributor's website. It is essential to follow the steps to the letter.

Skyline Mobility is not responsible for any adjustments, modifications, breakage or lack of stability caused by a poor quality hitch, shallow, too thin or without an adequate tightening mechanism. It is strongly recommended to use a hitch under the seat that's at least 15 cm deep to maximize roof support and stability. It is normal for the mica to be wrinkled after the initial assembly. Wrinkles will become less noticeable over time and with use.

### FABRIC MAINTENANCE

Do not use caustic, granular, or abrasive cleaners.

To keep the fabric in good condition and to delay the need for a deep, vigorous clean, the user should gently clean the fabric monthly with clean water to help prevent dirt from settling deep in the fabric.

Use a soft brush and lukewarm water with an appropriate fabric cleaner. Air dry only.

Never apply heat to the fabric. Always allow the fabric to dry completely before folding and storing it.

Always test the detergent on an inconspicuous area of the fabric before using it.

Mica may react to certain detergents. Always test the detergent on an inconspicuous area first.

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


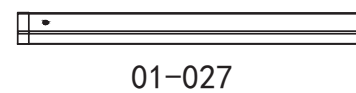
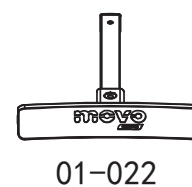
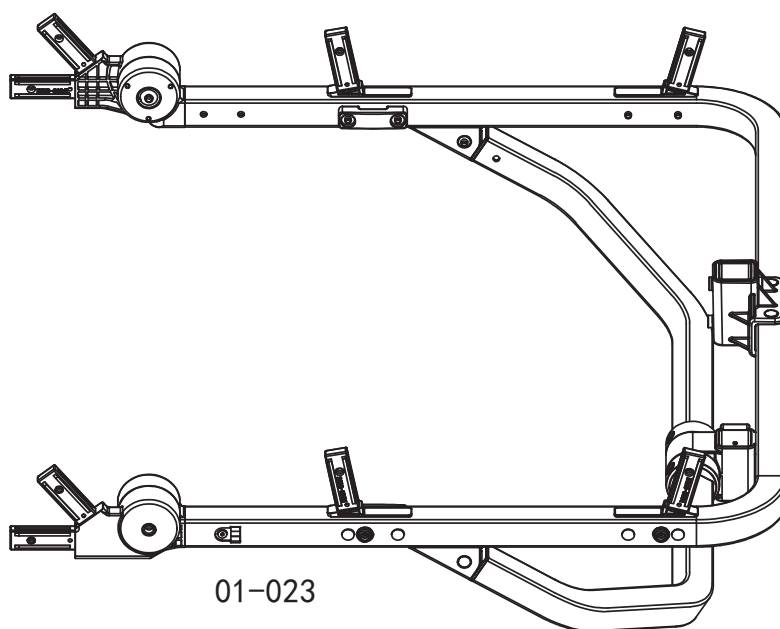
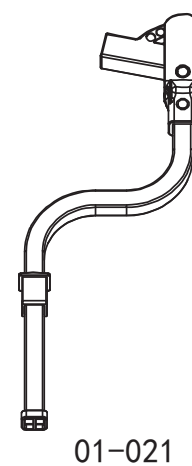
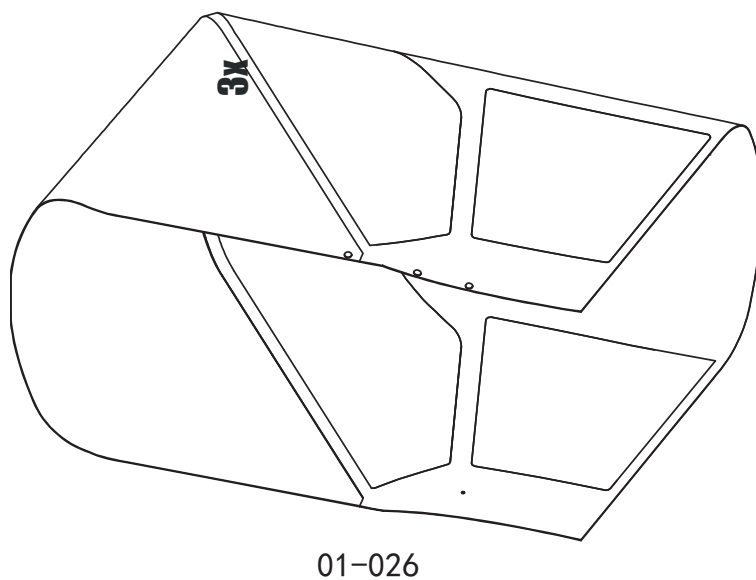
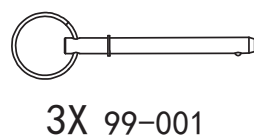
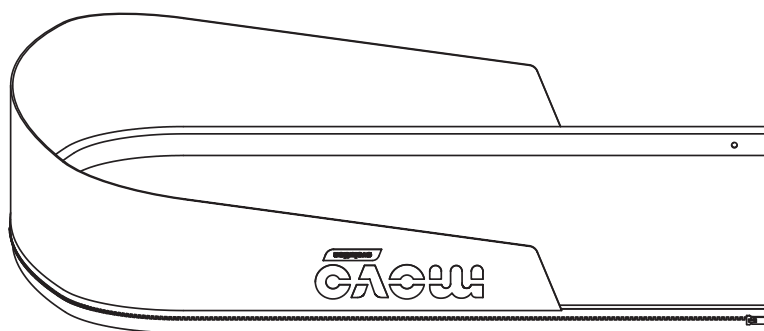
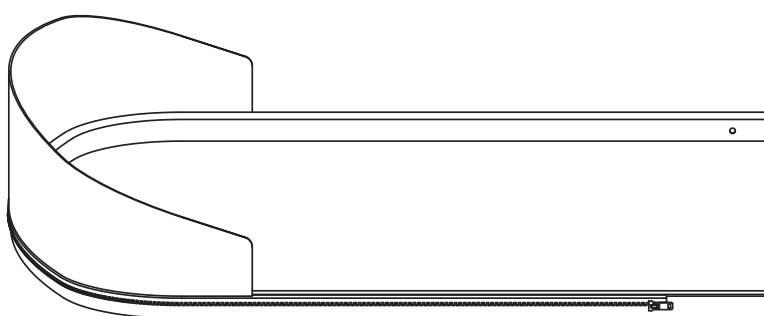
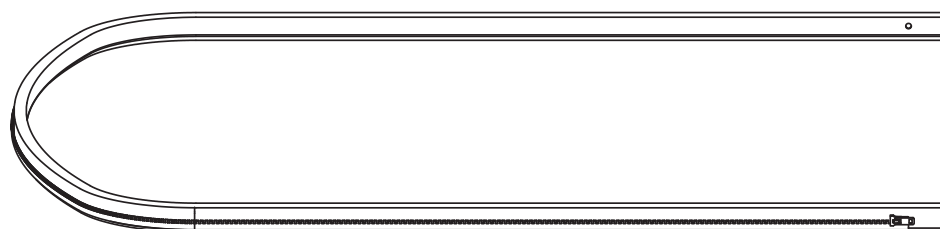
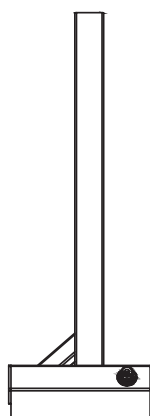
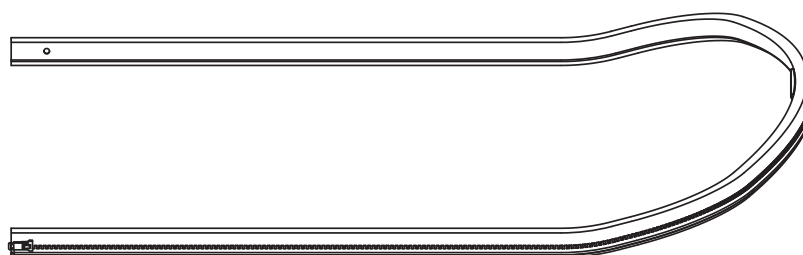
Ajust the seat to an adequate driving position.

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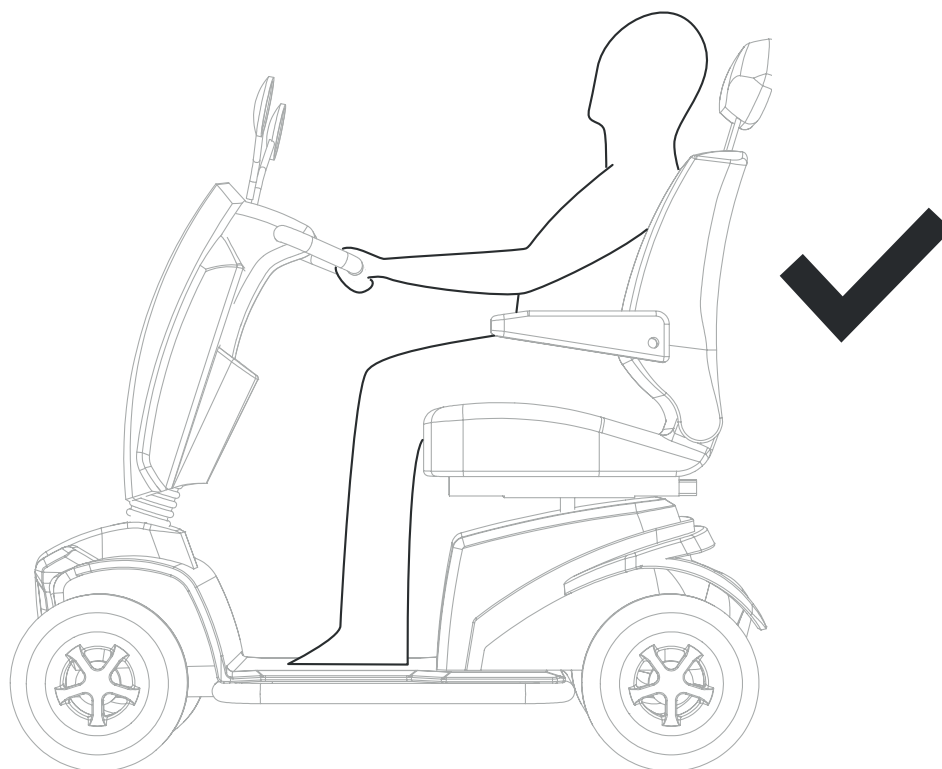
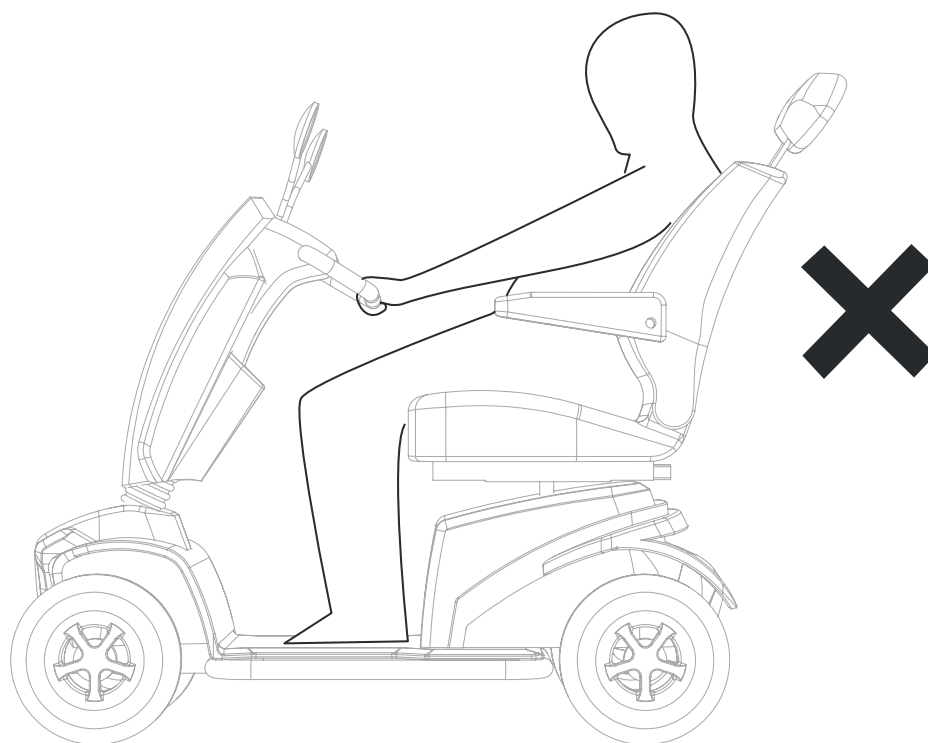
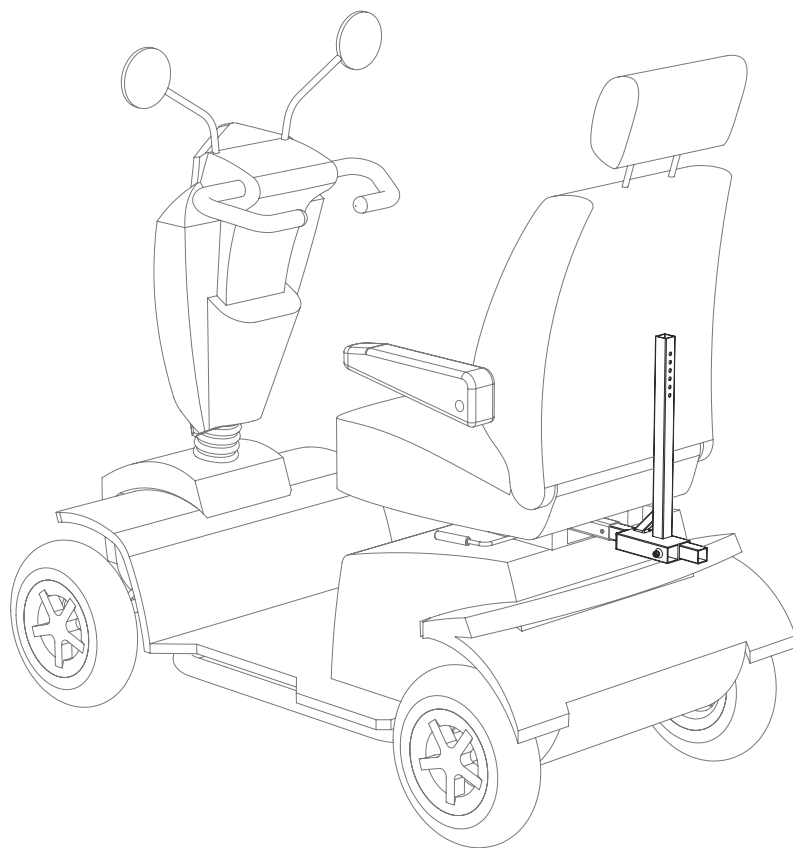
Slide 01-027 into the scooter seat assembly (accessory port) and secure it. Slide 01-020 over 01-027 and fix it. Make sure the diagonal bracket is positioned opposite the seat.

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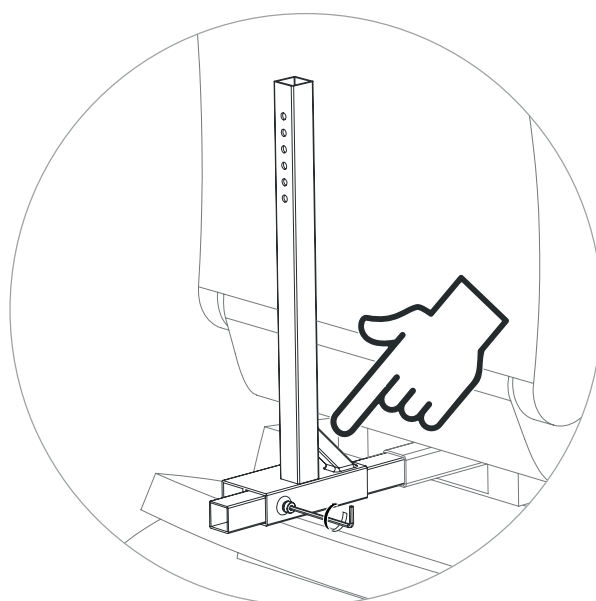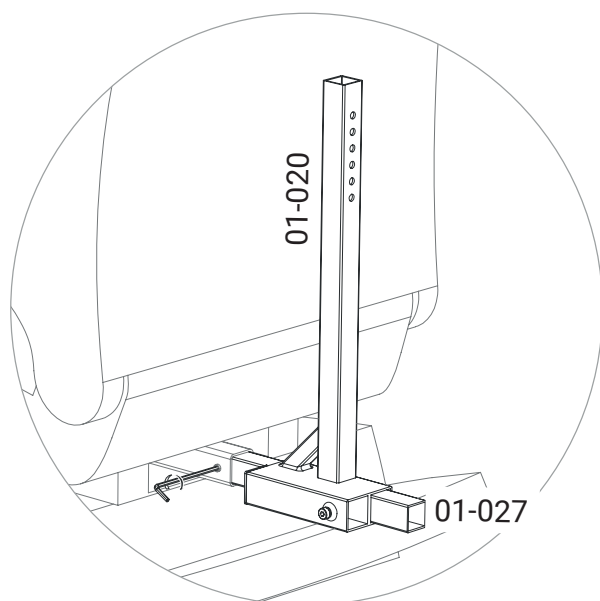
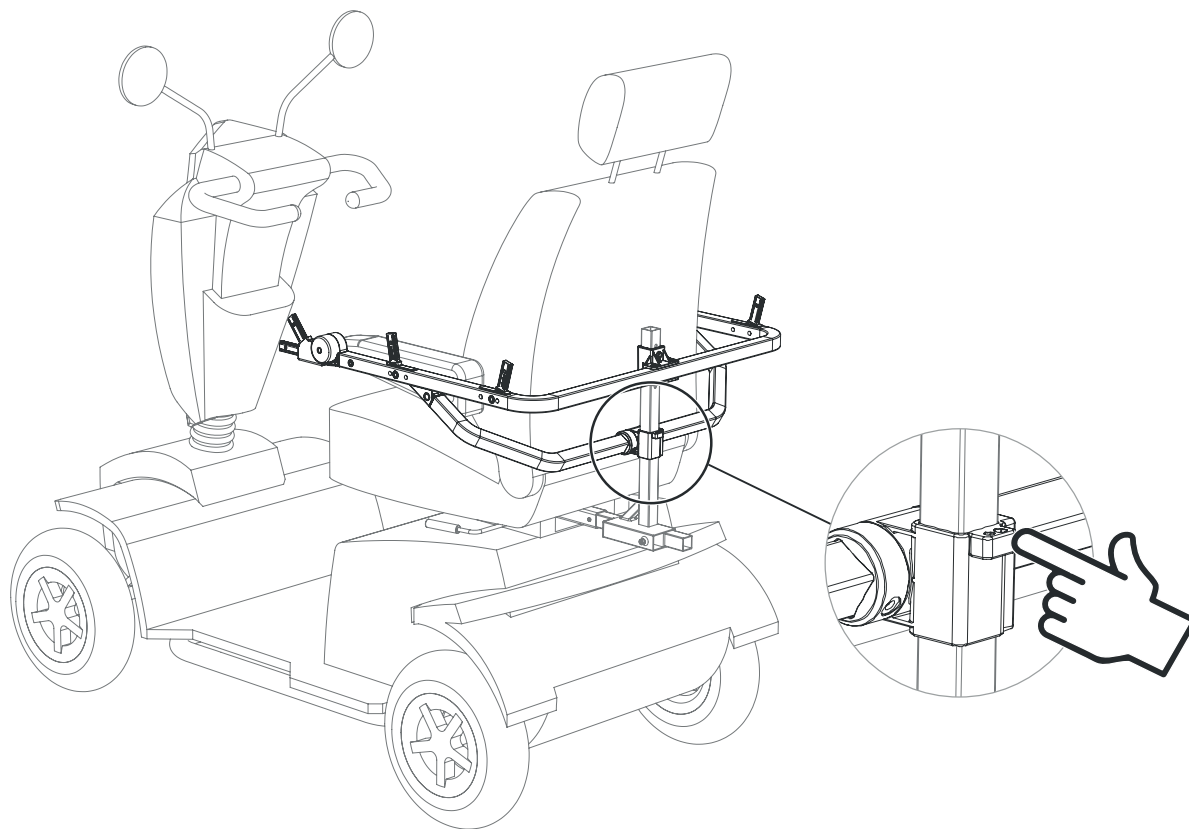
Slide 01-023 onto 01-020. Orient the pivoting part so that the allen key holder faces away from the scooter. Slide 01-023 vertically and align it to the armrests.

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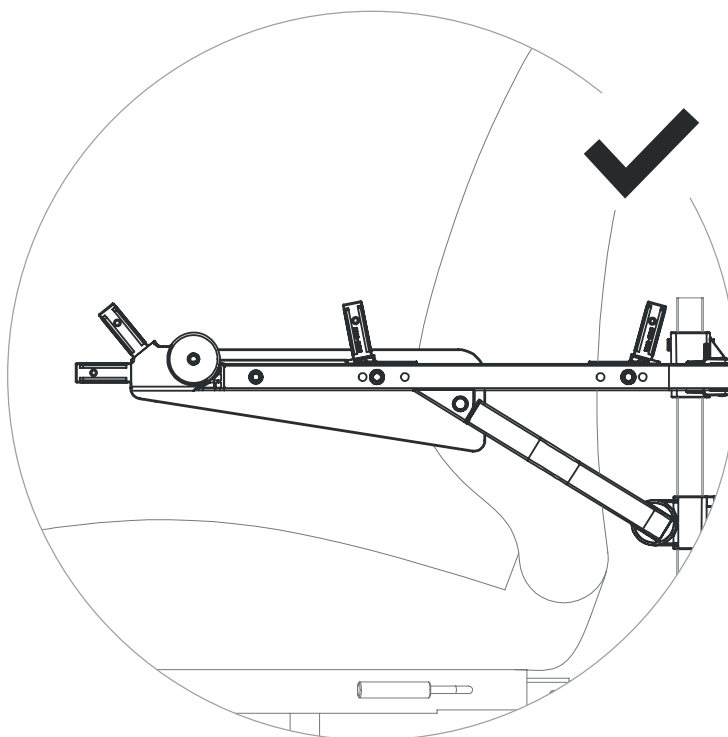
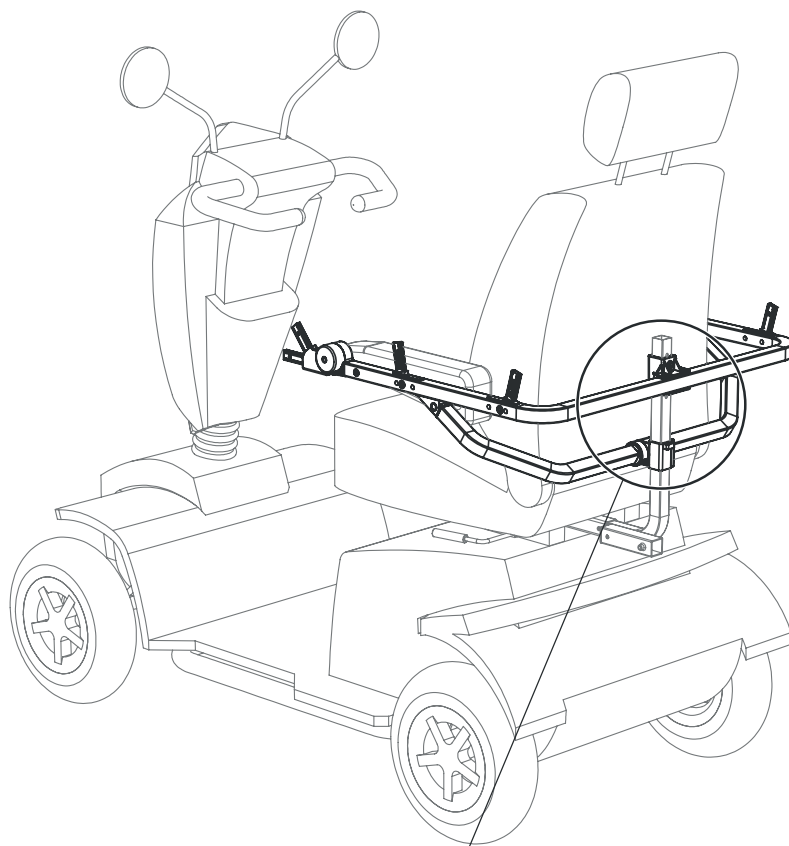
Temporarily insert 99-001 to hold 01-023 in place while you tighten the two M5 x 10mm screws. Once the screws are secure, remove 99-001 before proceeding to the next step.

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

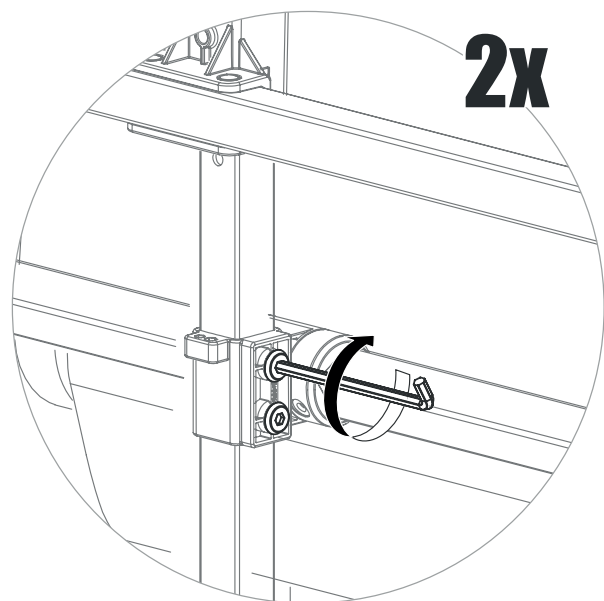
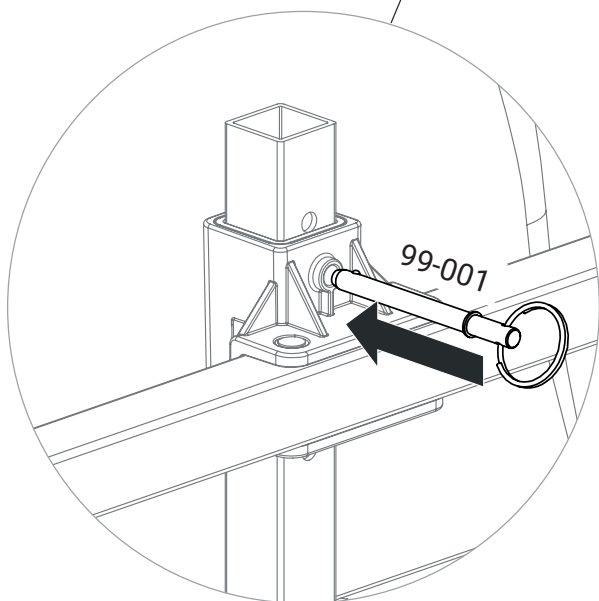
Slide 01-022 into 01-021 then slide the assembly into 01-020. Make sure to push the plastic sleeve all the way down into 01-020.

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
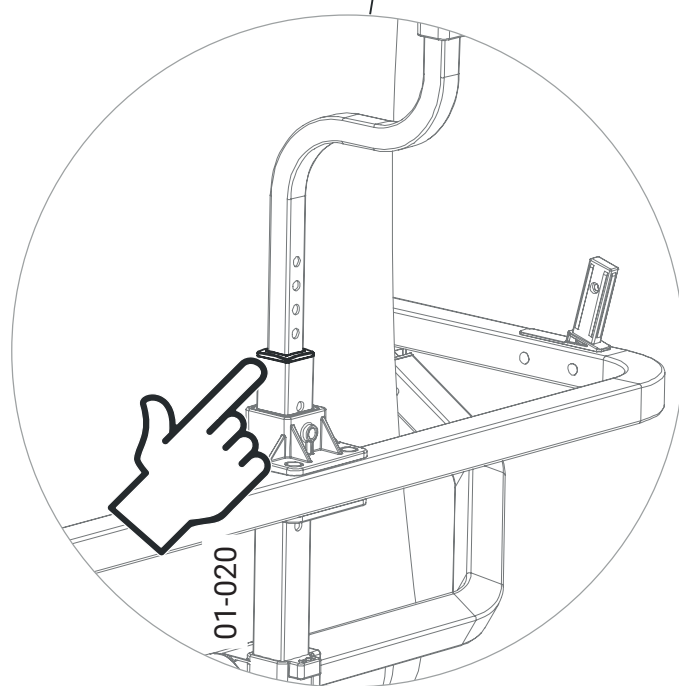
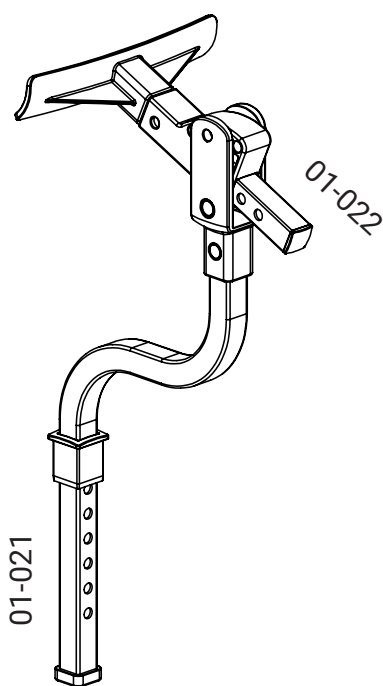
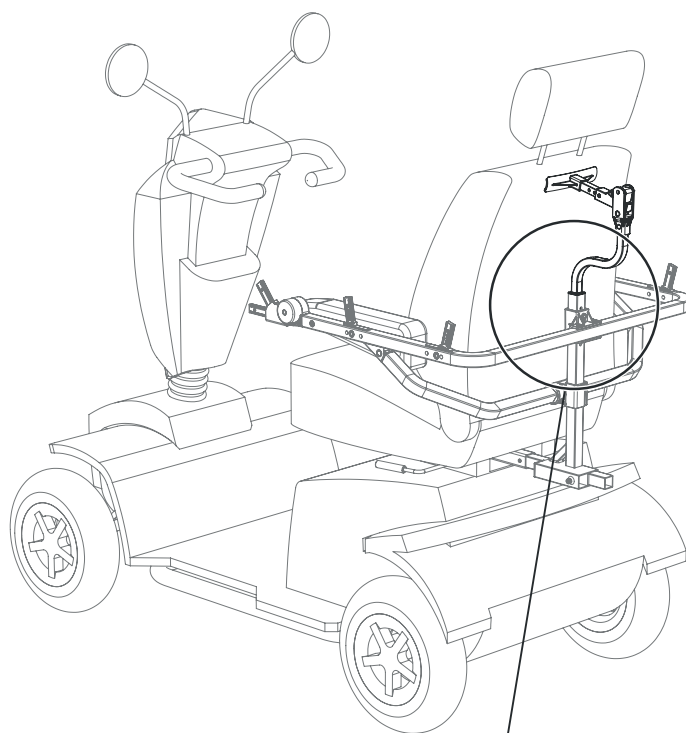
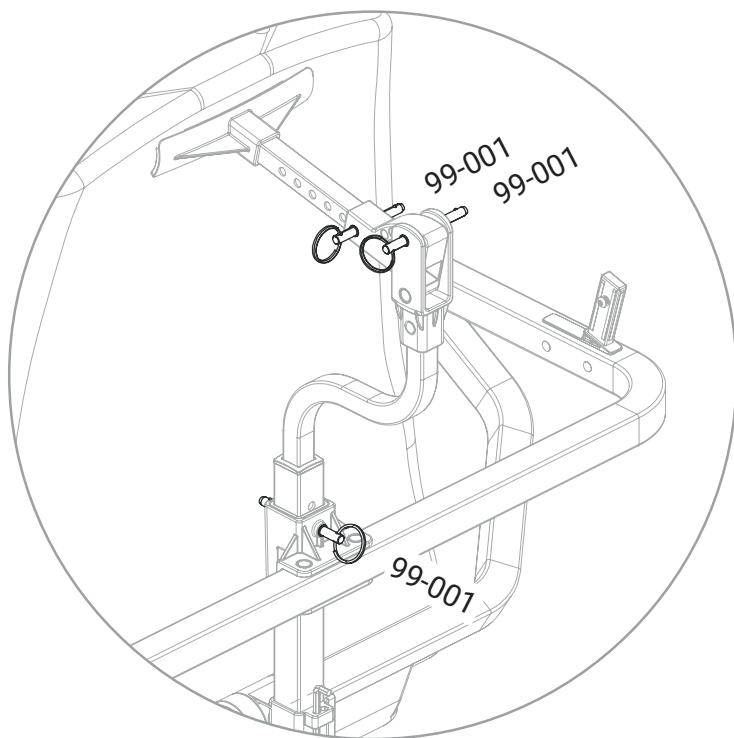
Adjust the height and angle of 01-021 so that 01-022 is pressing firmly near the top of the backrest. Do not press 01-022 against the headrest. Use three 99-001 pins to lock everything into position.

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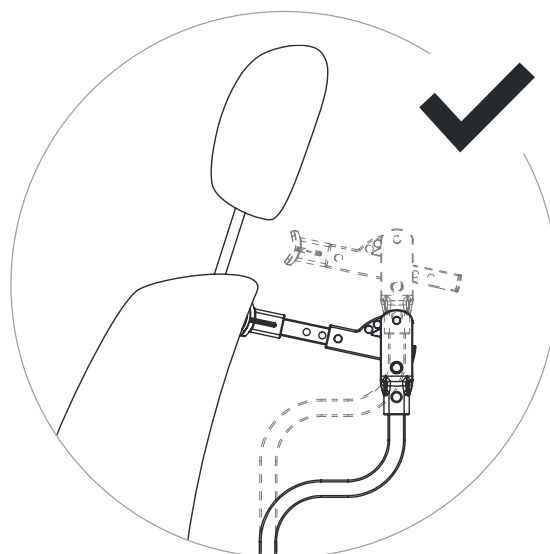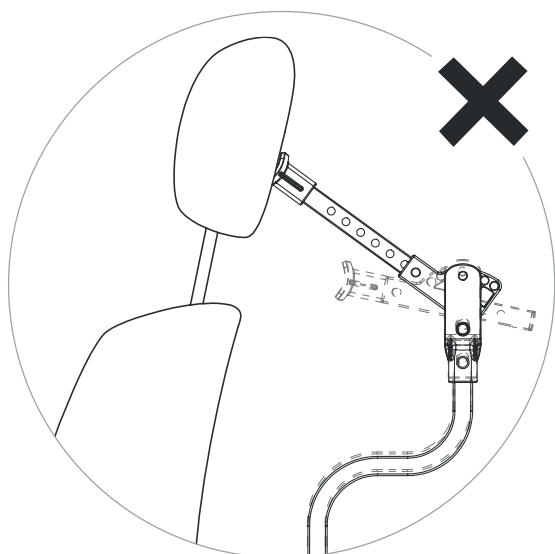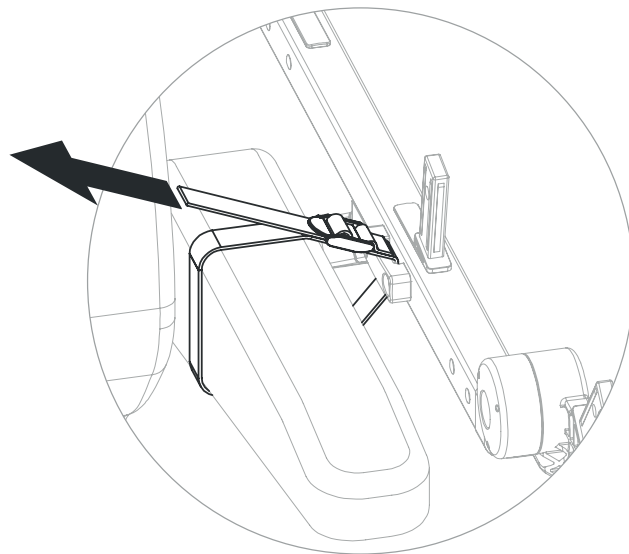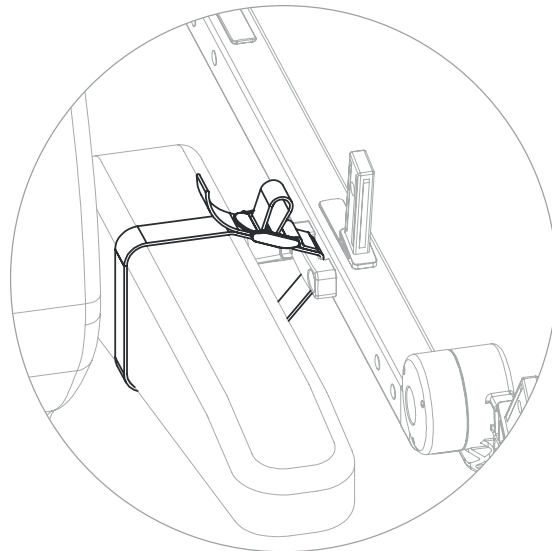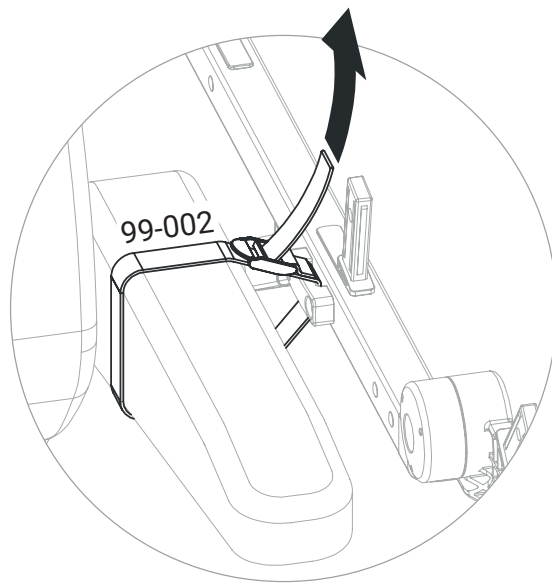
So far, 01-022 should be pressing against the backrest and 01-023 should be secured to the armrests using the 99-002 straps.

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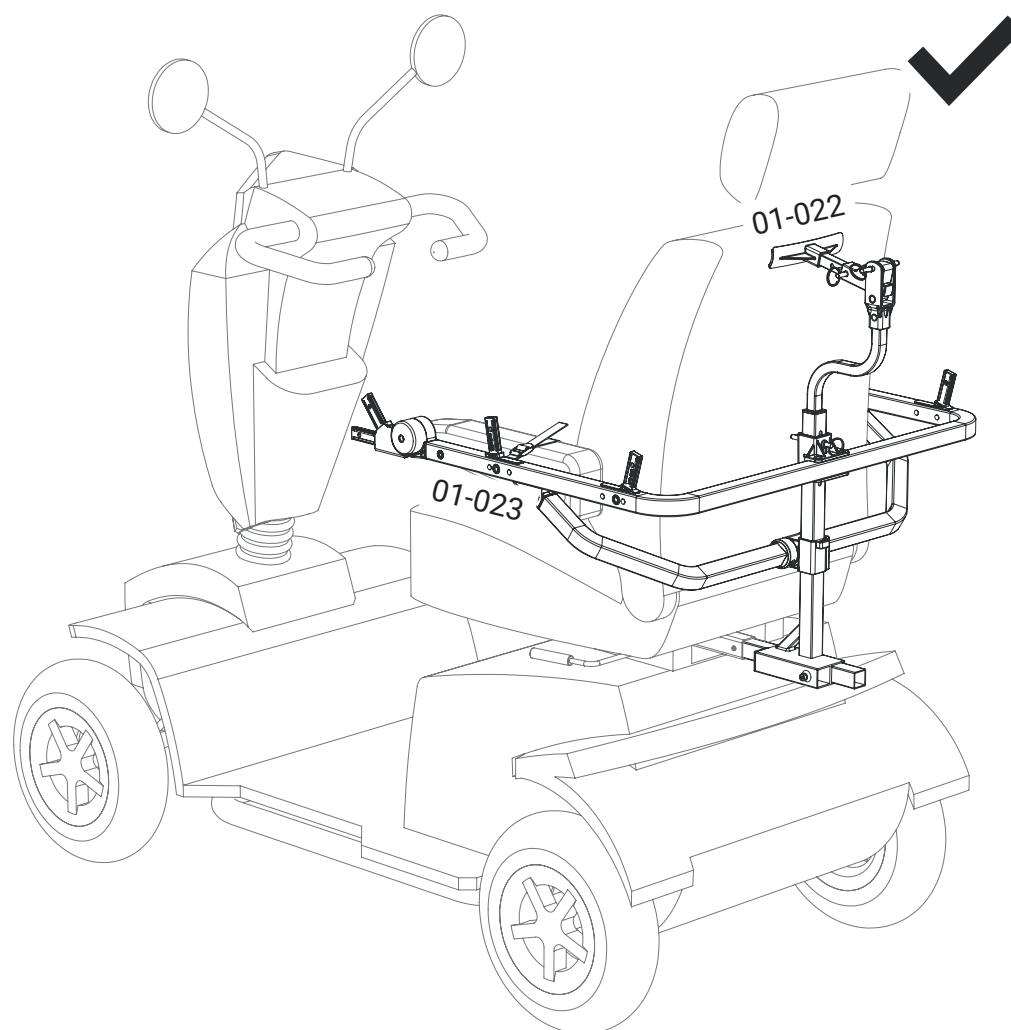
Refer to this diagram for the identification of the arches.

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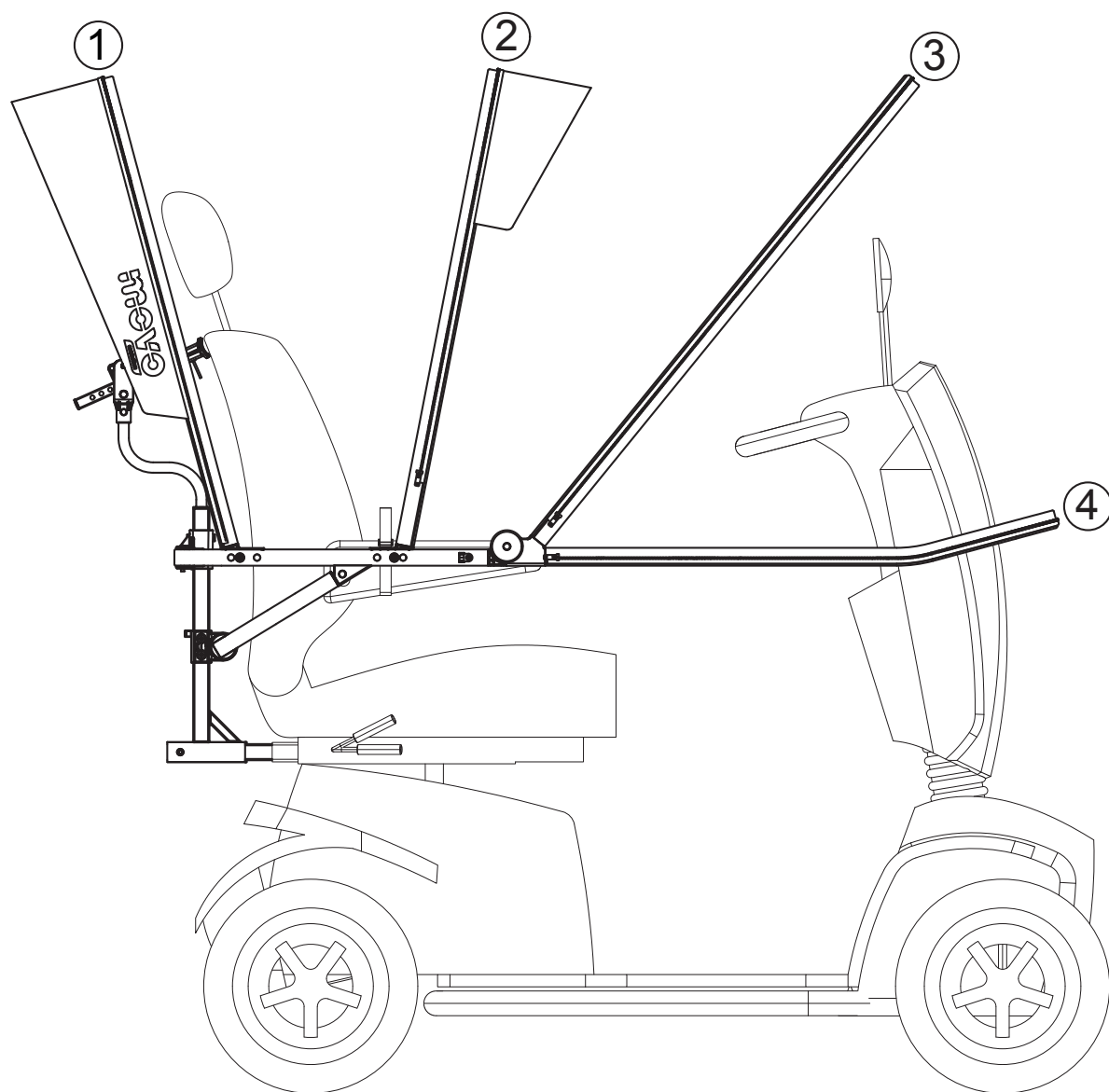
Slide arch 1 over the rear seals of 01-023 and make sure the zipper is facing the rear of the scooter. Assemble with two M5 x 10 mm screws.

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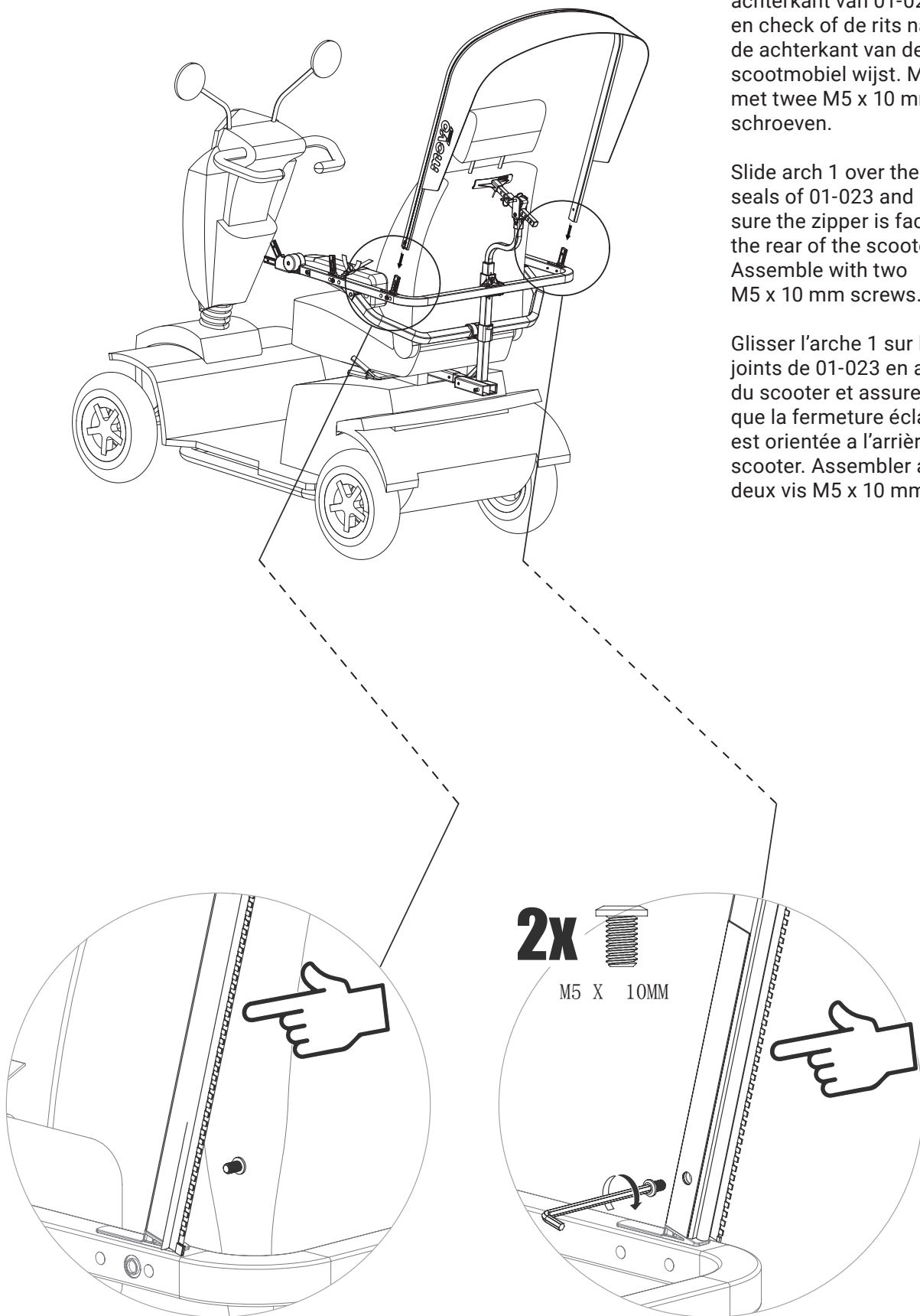
Slide arch 4 over the joints most in forward of 01-023 and make sure the zipper faces the floor and the MOVO logo is the right side up. Assemble with two M5 x 10 mm screws.

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

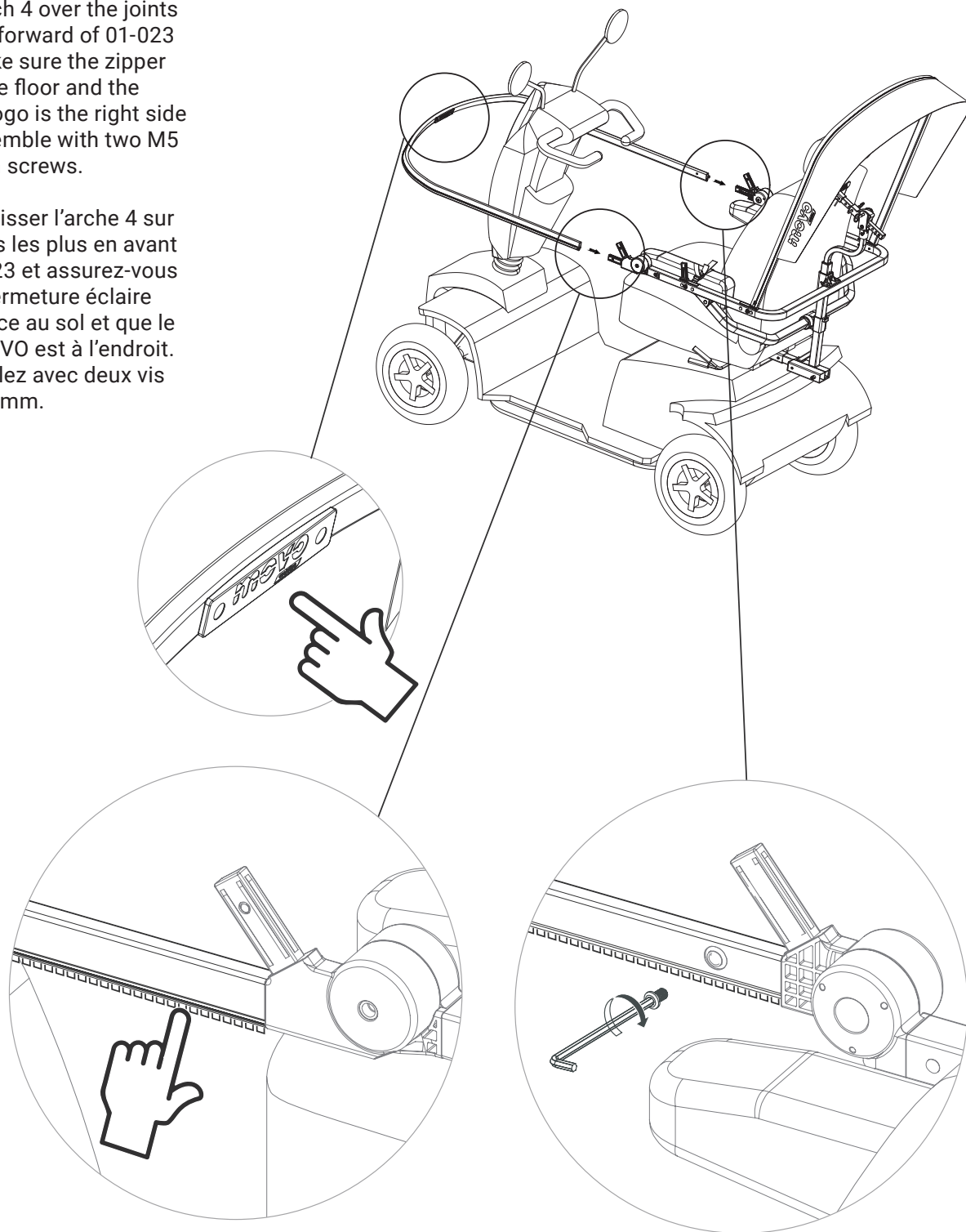
Slide arch 2 over the joints of 01-023 and make sure the zipper is facing the front of the scooter. Assemble with two M5 x 10 mm screws.

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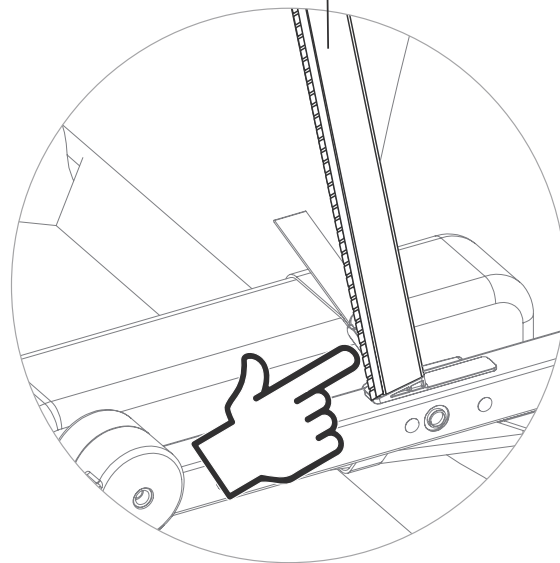
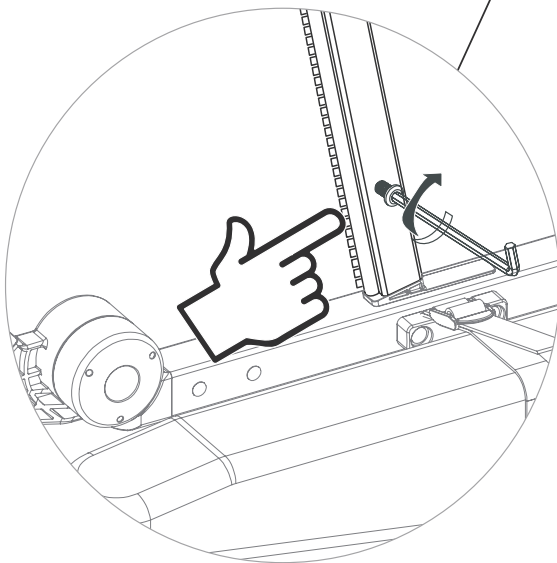
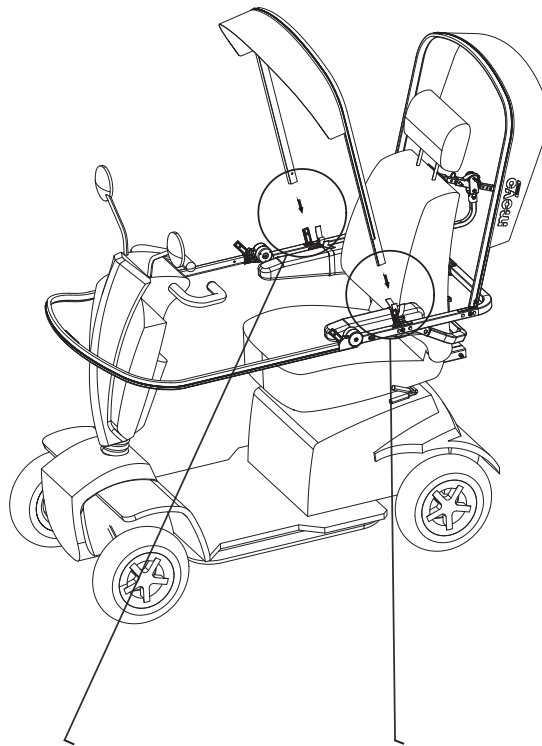
Slide arch 3 over the diagonal joints of 01-023 and make sure the zipper is facing the rear of the scooter. Assemble with two M5 x 10 mm screws.

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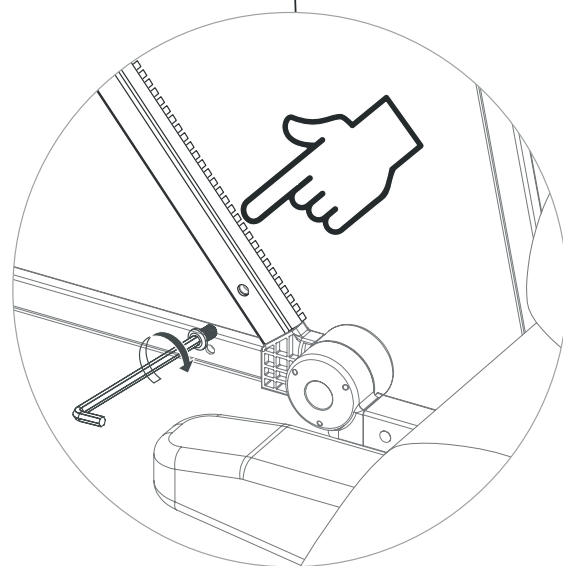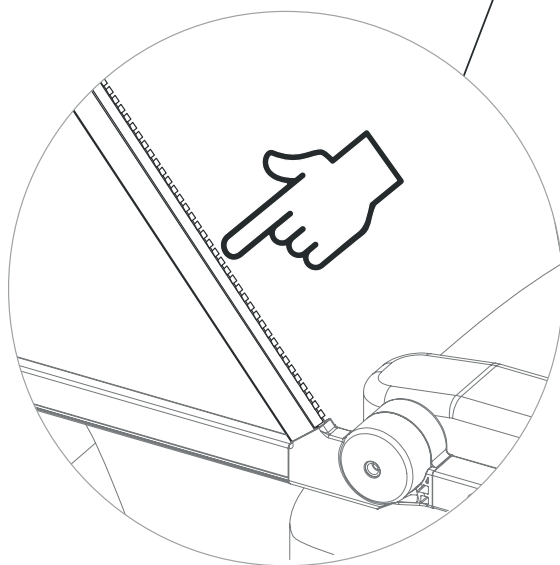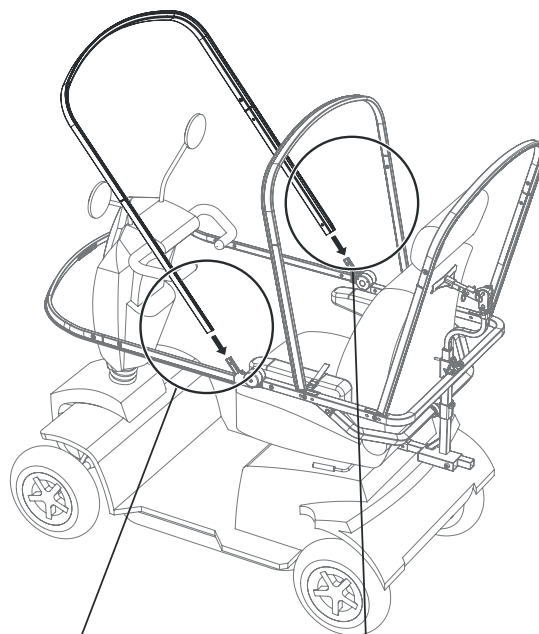
Before continuing, review the previous pages and make sure the zippers are oriented correctly. Then make sure that the head of the zippers are in the places indicated by a circle.

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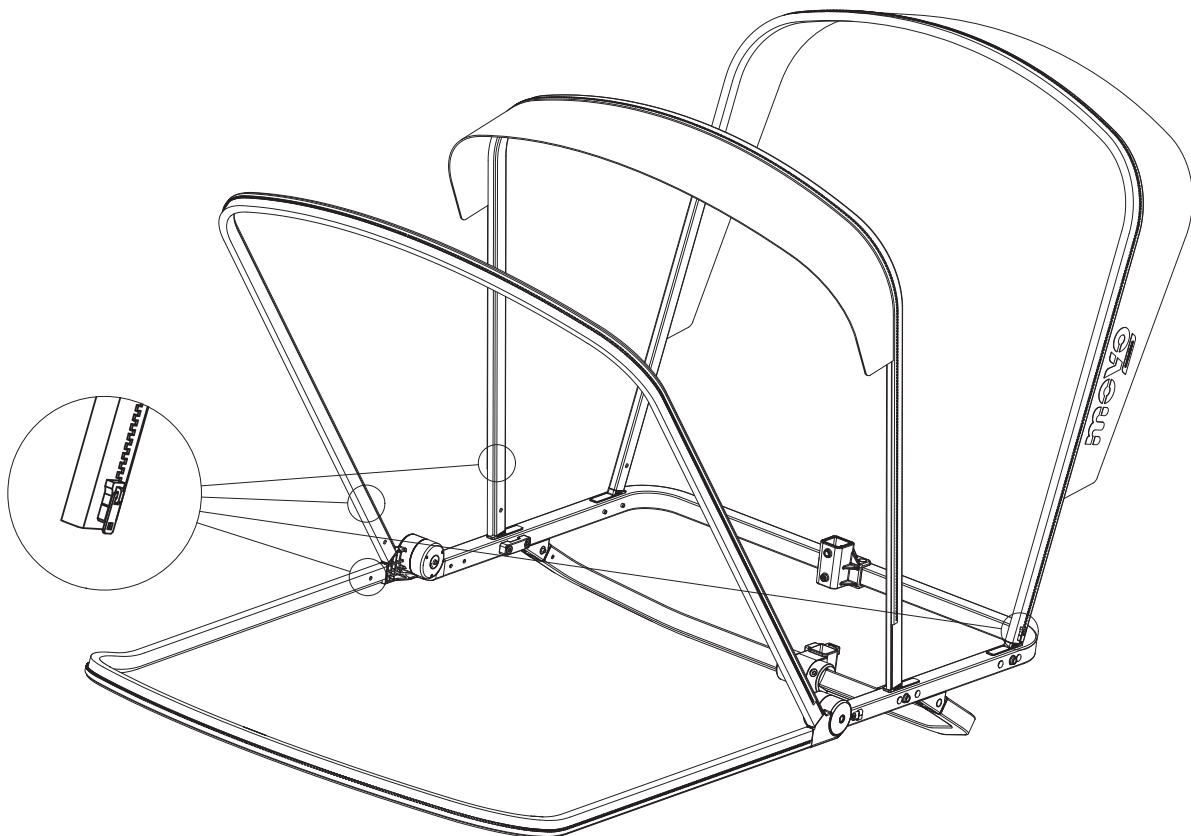
Unfold and lay the canvas on the roof frame. Make sure the gray textile part is facing the back and the transparent mica is on the front.

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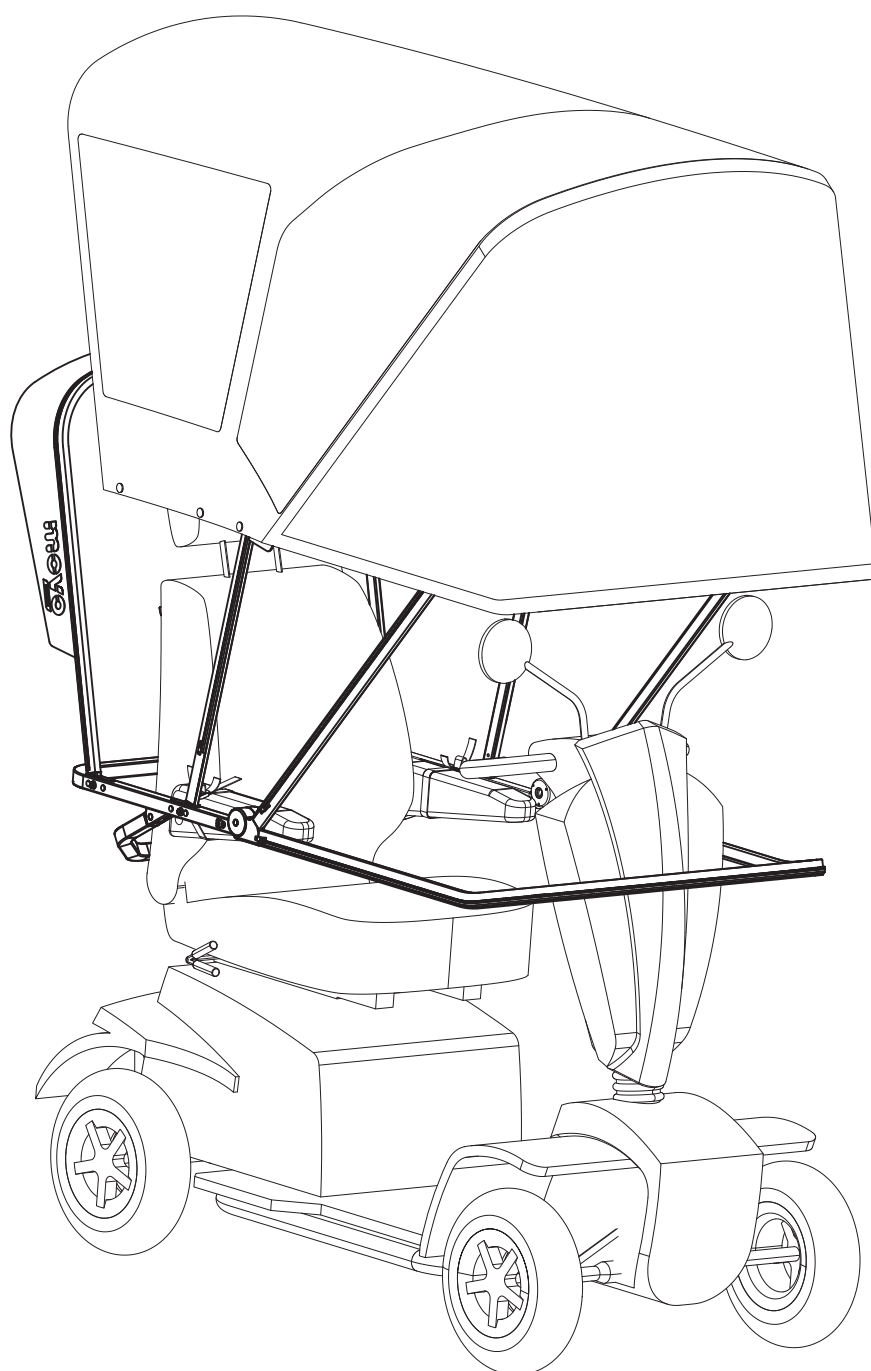
Connect the zippers on the inside of the canvas and close them gently in the order shown below.

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

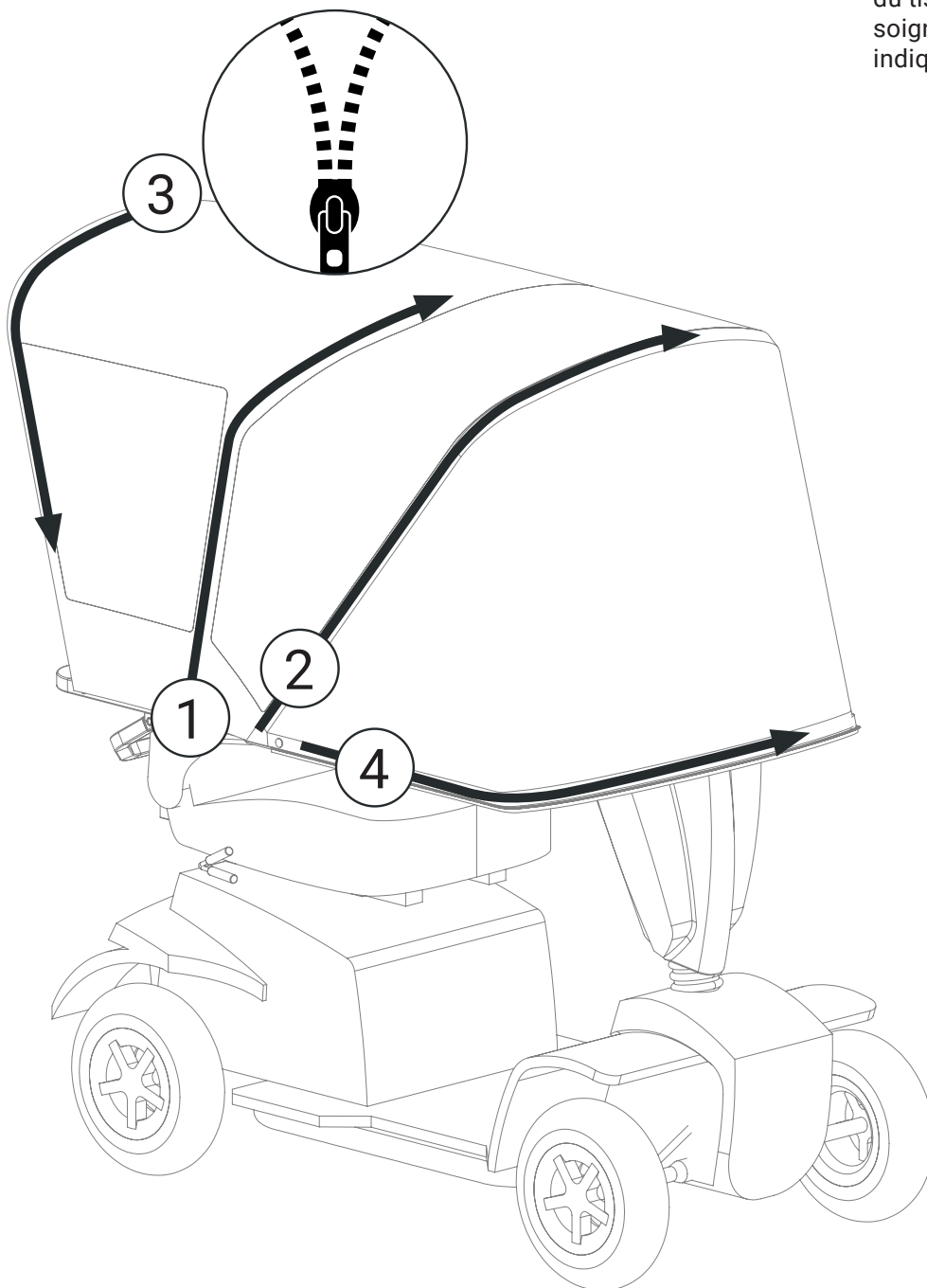
Locate the 3 snaps on each side of the canvas. Start by threading the hook at the back, then push in the two snaps. Repeat on the other side.

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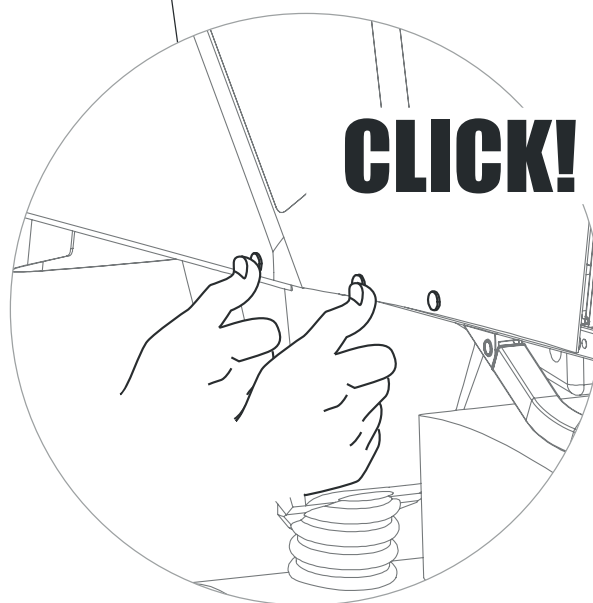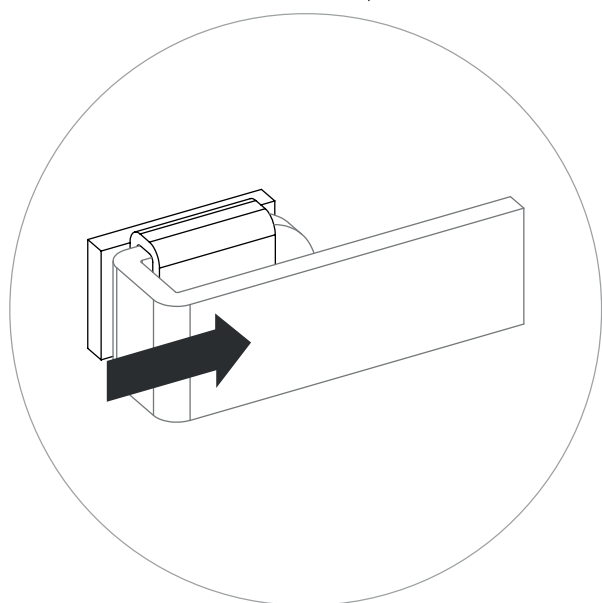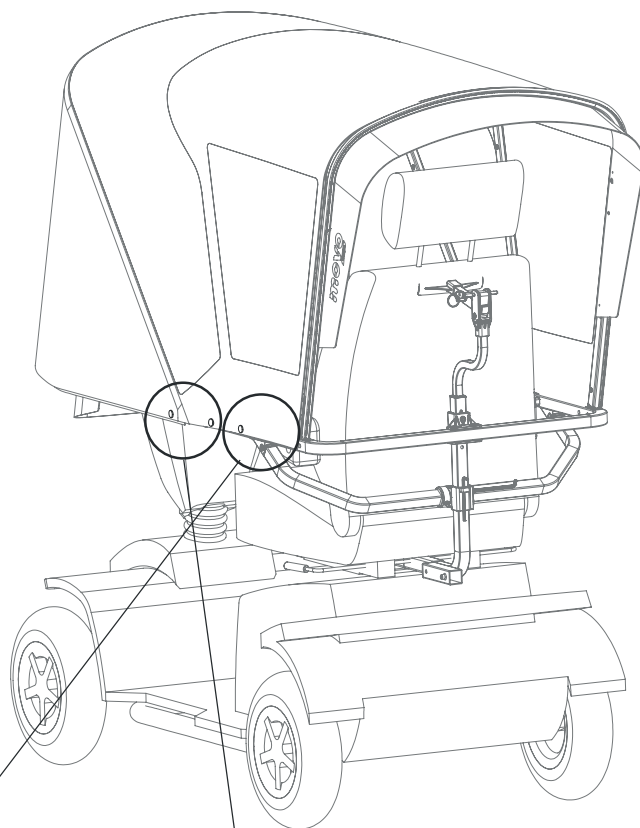
Lift the front section of the roof with a brisk motion, placing one hand under the front arch and one hand on the rear arch for more control.

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
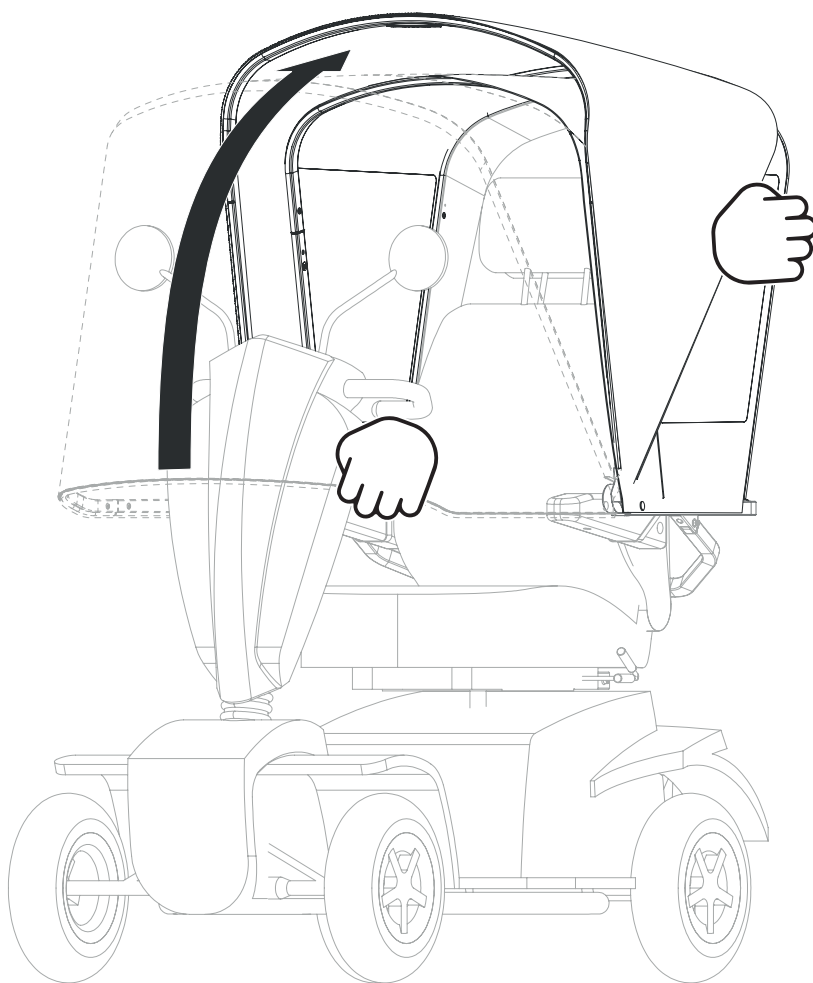
The roof should be parallel to the ground and stable when driving. If not, refer to the following solutions (p.29 and 30). **Users of short-seat scooters (< 51 cm) should refer to page 29 only.**

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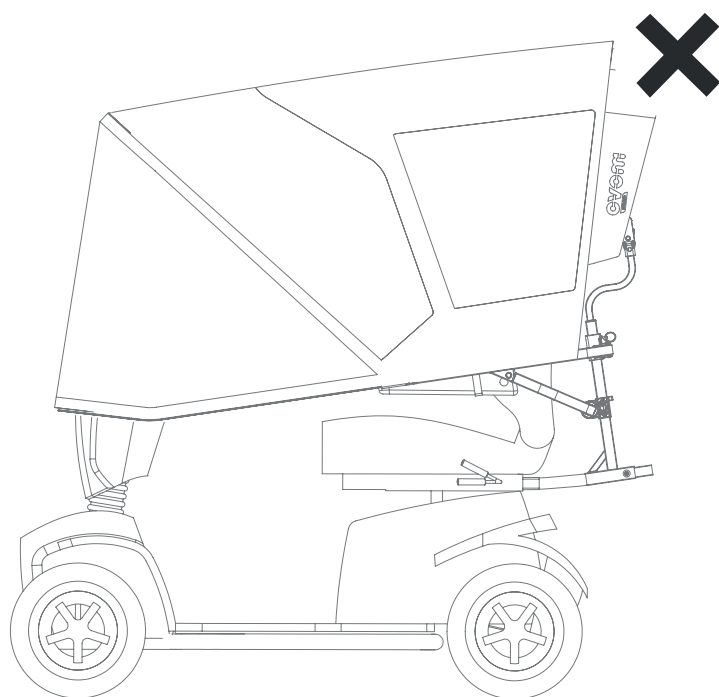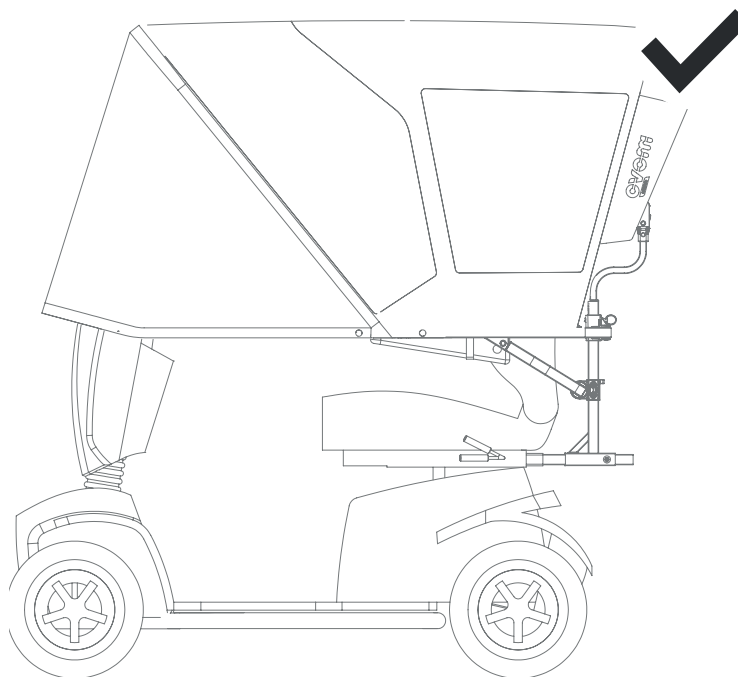
**Important for users of short-seat scooters (< 51 cm).** Without removing pins 99-001, unscrew the two M5 screws 01-023. Lift the lower tube with force and bring it **as close as possible** to the upper tube which is at the level of the armrests. Screw firmly in place.

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

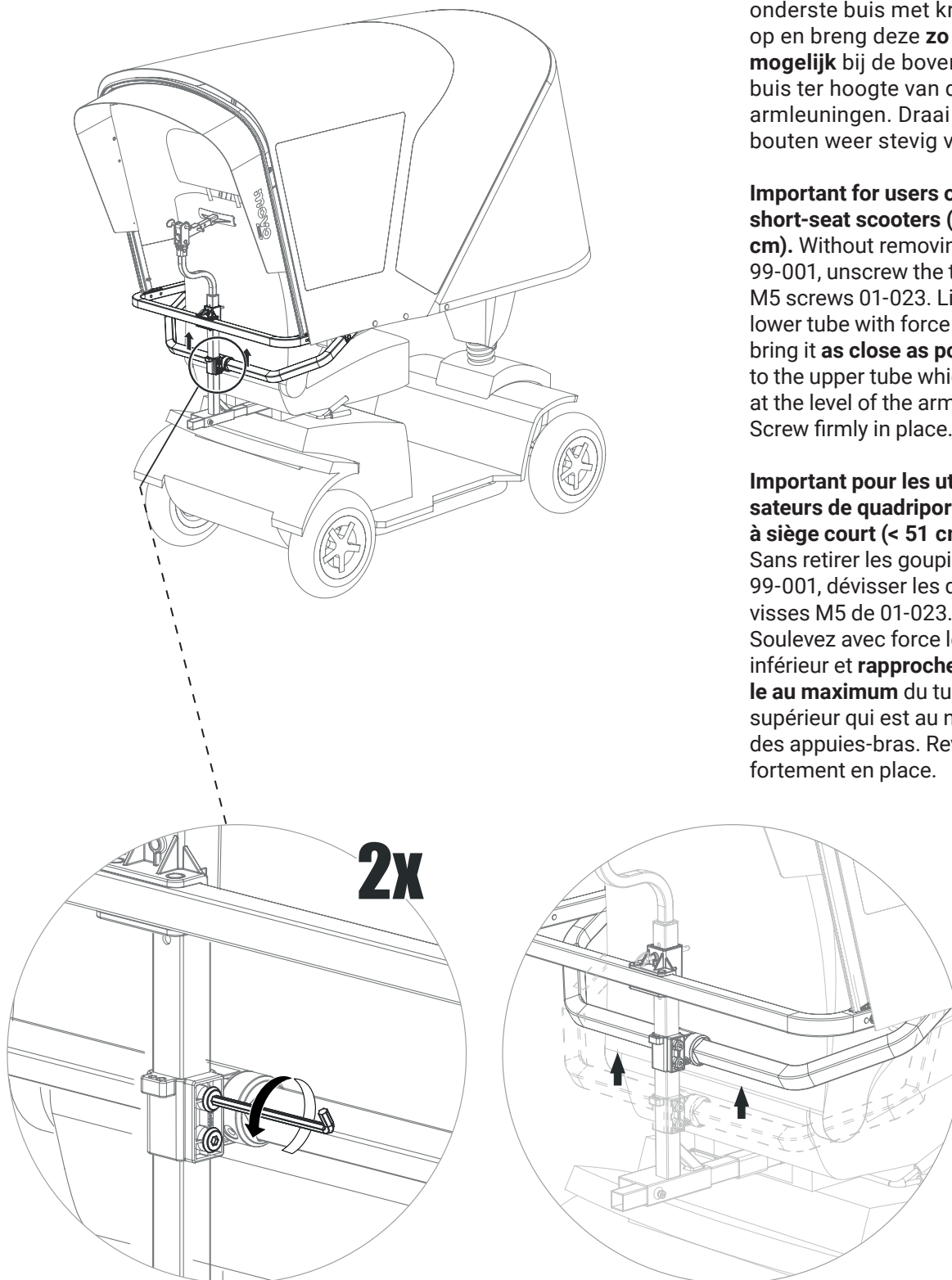
Loosen the hitch and push 01-020 towards the front of the scooter. This will ensure that 01-022 will be better pressed against the backrest. Tighten.

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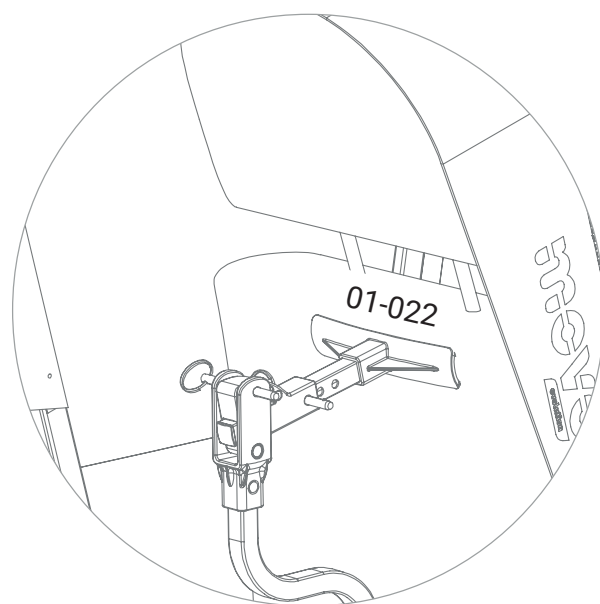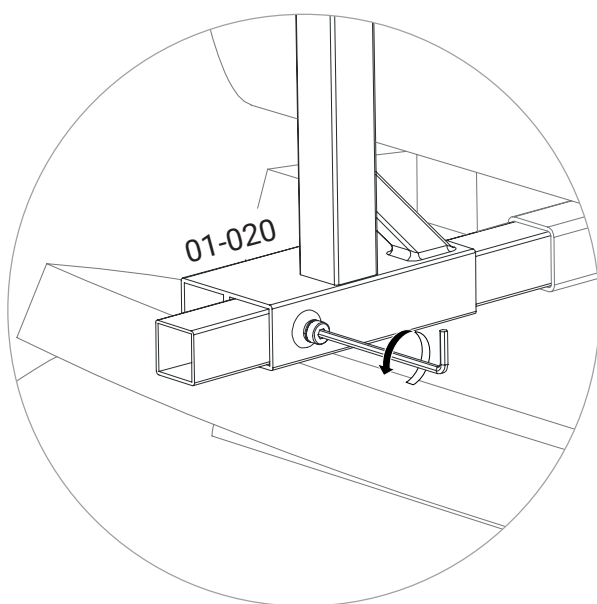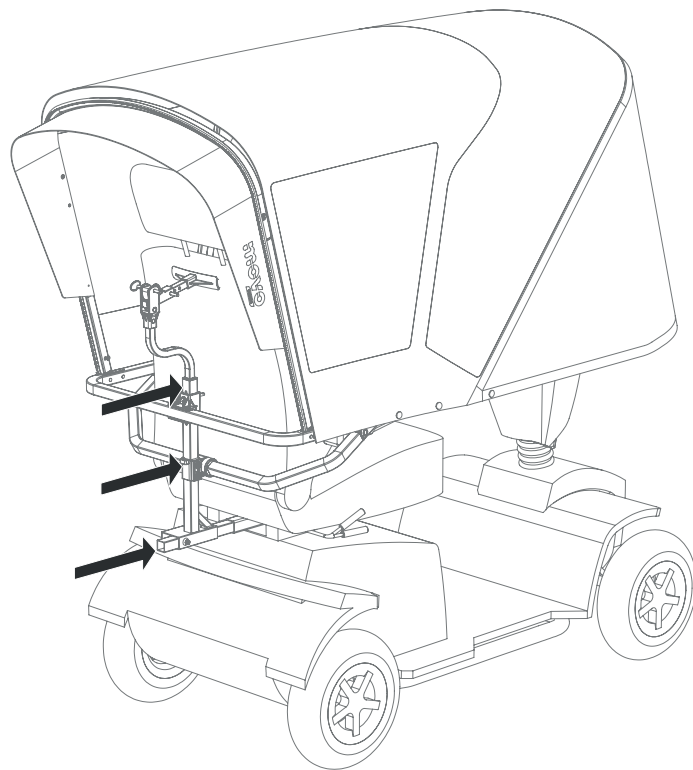Congratulations! You have finished assembling your Movo Evolution roof.  
Don't forget to visit [www.skylinemobility.com](http://www.skylinemobility.com) for complementary accessories.

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